

Application Note

Update Procedure for the eego amplifier MS Windows Driver to version 1.4.41

Objective

This application note describes the process for updating the windows drivers for eego amplifiers (EE-22x, EE-511, EE-411, EE-410) by using the zip file DDE-OP-3238.

Update procedure

1. Stop all applications that use eego amplifiers like e.g. eego, neo, or visor2.
2. From **Settings -> Apps -> Installed apps** search for **Eego64 Amplifier Driver** and select Uninstall.
3. Unzip the archive file **DDE-OP-3238rev1.4.41 eego amplifier Windows Driver**.
4. Execute the unpacked file **EegoDriver_1.4.41.msi**.
5. Start your application that uses the eego amplifiers
6. Check that eego amplifiers are correctly detected and available in your application.